

We build only fine quality seating ...



... for those who only want the finest.

April 5, 2007

SERVICE BULLETIN

Subject: Locking Quick-Release Slot Bases

Re: Locking Bracket Obstructions

In order for the notched base floor plate to properly engage and securely lock into the cabinet mounted locking bracket, the bracket must be kept clean and clear of all foreign debris. Base must be securely locked into the bracket before stool is used. Failure to have stool base properly locked could result in stool tipping and possible serious injury.



Clear all debris, using Bracket Release Key that was provided.

Any base or bracket that fails to securely lock should be immediately removed from service.

For further information or assistance, please contact our Customer Service Dept at (800) 323-2234

Gasser Chair Company, Inc.

4136 Loganway • Youngstown, Ohio 44505-1797 • Tel: 330-759-2234 • Fax: 330-759-2227